

KRUUSE Muzzle Guide



BUSTER Extreme Dog Muzzle

- Easy to fit
- Tough and durable, yet flexible and soft
- Variable collar adjustment
- Safe and strong dog muzzle with strategic addition of struts at front nose area to eliminate all possibilities of biting
- Dog friendliness in terms of feeding (nibble and water) and panting

The BUSTER muzzle is a comfortable dog muzzle designed to protect against bites.

The safety strapping ensures that the dog muzzle will always remain in place and it features two additional points of secure attachment. The loop at the bottom is designed to attach to your dog's regular collar.

The muzzle has a secure metal buckle with pre-holed webbing to ensure easy adjustment and a good fit.

The KRUUSE dog muzzle is made from injection moulded thermo plastic rubber, which is extremely tough and durable.

The neoprene padding inside of the webbing straps ensures that it is snug and comfortable for the dog to wear.

The dog can pant, drink and be given treats to reward a more placid attitude to life.



Tough and durable material for security purpose



Allows drinking, panting and vomiting



Inhibits biting, chewing and barking



Difficult for dog to dislodge



Sizing

The best way to choose the model size is to use the weight/breed guidelines, as long as your dog is not over or underweight.

If your dog is not at an ideal weight, or is between sizes, we recommend that you use these measuring instructions:

- Use a soft measuring tape, measure the length of your dog's snout from the end of his nose to the point where the snout starts to slope upwards to the eyes as shown on the drawing
- Then, measure around the widest part of your dog's snout while his mouth is closed

Weight/Breed Guideline

Cat. No	Size	Approx. Weight of dog (kg/lbs)	Example Breeds (Avg. Sized adults)	Snout Length (cm / inch)	Snout Circumference (cm / inch)	Adjustable Neck Circumference (cm/ Inch)
272351	Size 1	4.5 to 7 kg 10 to 15 lbs	Bischoon Frise, Dachshund, Miniature Pinscher, Large Chihuahua, Jack Russell	5.5 cm / 2.1 in	21.5 cm / 8.4 in	20 cm / 7.8 in to 39 cm / 15.3 in
272352	Size 2	5.5 to 11 kg 12 to 25 lbs	Border Terrier, Jack Russell, Cocker Spaniel, Small Beagle, Miniature Poodle, Sheltie	6.5 cm / 2.5 in	27 cm / 10.6 in	21 cm / 8.2 in to 44 cm / 17.3 in
272353	Size 3	11 to 20 kg 25 to 45 lbs	Border Collie, Large Beagle, Corgi, Vizsla, Standard Poodle	7 cm / 2.7 in	30 cm / 11.8 in	27 cm / 10.6 in to 52 cm / 20.4 in
272354	Size 4	18 to 29 kg 40 to 65 lbs	Springer Spaniel, Dalmatian, Pointers, Small Pit Bull, Cattle Dog	8 cm / 3.1 in	34 cm / 13.3 in	29 cm / 11.4 in to 58 cm / 22.8 in
272355	Size 5	27 to 40 kg 60 to 90 lbs	Labrador, Golden Retriever, Doberman, German Shepherd, Greyhound, Weimaraner, Husky, Large Pit Bull, Boxer	9.2 cm / 3.6 in	39 cm / 15.3 in	34 cm / 13.3 in to 60 cm / 23.6 in
272356	Size 6	36 to 68 kg 80 to 150 lbs	Great Dane, Mastiff, Newfoundland, Rottweiler, Giant Schnauzer, X-Large Pit Bull, Large German Shepherd	10.5 cm / 4.1 in	43 cm / 16.9 in	36 cm / 14.1 in to 71 cm / 27.9 in

The BUSTER Extreme Dog Muzzle is washable and can be sterilized. Do not use if damaged. Do not leave your dog muzzled without supervision.



Soft neoprene padding for extra comfort



Overhead strap connects directly to the rear strap



Allows panting and treating



BUSTER Easy-ID Dog Nylon Muzzle

Line of high-quality colour coded nylon muzzles that have a side easy-release snap buckle


- High quality water-resistant
- Colour coded for easy recognition
- Extra padding around the nose
- Muzzles fit securely with adjusting straps for a snug fit
- Lightweight nylon
- Reinforced stitching to ensure long-lasting use
- Available in 10 sizes from 2XS to 5XL



**COLOUR CODED
FOR EASY SIZE IDENTIFICATION**

Selecting the right muzzle for your dog

- Use a string to measure around your dog's nose at the position shown
- Place your index finger under the string to ensure enough space for the dog to be comfortable. Do not tighten the string; it should be free to move around the dog's nose
- Lay the string along a ruler to get appropriate length, then select the proper muzzle
The muzzle must not be tight. It is better a little loose than too tight

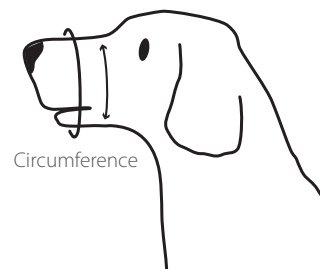

Weight/Breed Guideline

Cat. No	Size	Patient Weight Range (kg)	Patient Weight Range (lbs)	Snout Circumference (cm)	Snout Circumference (")	Colour	Breeds
279465	2XS	< 2 kg	< 5 lbs	9 cm	3½"		Chihuahua
279466	XS	2-4 kg	5-10 lbs	11.5 cm	4½"		Toy Poodle
279467	S	4-11 kg	10-25 lbs	13 cm	5¼"		Dachshound
279468	M	11-18 kg	25-40 lbs	15 cm	6"		Cocker Spaniel
279469	L	18-31 kg	40-70 lbs	16.5 cm	6½"		Bull Terrier
279470	XL	31-45 kg	70-100 lbs	20 cm	8"		Doberman
279471	2XL	45-68 kg	100-150 lbs	25 cm	10"		German Shepherd
279472	3XL	45-54 kg	100-120 lbs	28 cm	11"		Labrador Retriever
279473	4XL	54-68 kg	120-150 lbs	30 cm	12"		Great Dane
279475	5XL	> 68 kg	> 150 lbs	35 cm	14"		Giant Breeds
279477	XXS-5XL	Set of 10 BUSTER Easy-ID Dog Nylon Muzzles				Ass.	All
279487	S-2XL	Set of 5 BUSTER Easy-ID Dog Nylon Muzzles				Ass.	All
279489	XS – 2XL	Set of 6 BUSTER Easy-ID Dog Nylon Muzzles + 1 BUSTER muzzle for cat (size M)				Ass.	All



BUSTER Dog Nylon Muzzle


- Made from polypropylene, strong and comfortable material
- Easy to fit securely, strong with handy snap-lock
- Machine washable at 30° C

Suitable for dog breeds such as

- Size 0 Yorkshire Terrier, Mini Poodle
- Size 1 West Highland White Terrier, Jack Russel, Dachshund
- Size 2 Border Collie, Beagle, Cocker Spaniel
- Size 3 Labrador Retriever, German Shepherd, Dobermann
- Size 4 Wolfhounds, Great Danes, Pyrenean
- Size 5 Rottweiler
- Size 6 Extra long for large Wolfhound, Great Dane
- Size 7 Extra long for large Rottweiler
- Size 8 Boxer, special model for short-nosed dog breeds



Size	KRUUSE Cat. No	Circumference	Top Line	Height	Jawline
Measurements in cm (")					
Size 0	279395	13.0 (5.1")	3.0 (1.1")	6.0 (2.3")	6.0 (2.3")
Size 1	279396	14.0 (5.5")	4.0 (1.5")	6.0 (2.3")	8.0 (3.1")
Size 2	279397	17.0 (6.6")	5.5 (2.1")	7.0 (2.7")	9.0 (3.5")
Size 3	279398	20.0 (7.8")	6.0 (2.3")	9.0 (3.5")	12.0 (4.7")
Size 4	279399	25.0 (9.8")	8.0 (3.1")	11.0 (4.3")	15.0 (5.9")
Size 5	279400	22.5 (8.8")	7.0 (2.7")	10.0 (3.9")	14.0 (5.5")
Size 6	279401	30.0 (11.8")	8.0 (3.1")	14.0 (5.5")	14.0 (5.5")
Size 7	279402	28.0 (11")	6.0 (2.3")	13.0 (5.1")	12.0 (4.7")
Size 8	279403	31.0 (12.2")	4.0 (1.5")	14.0 (5.5")	9.0 (3.5")

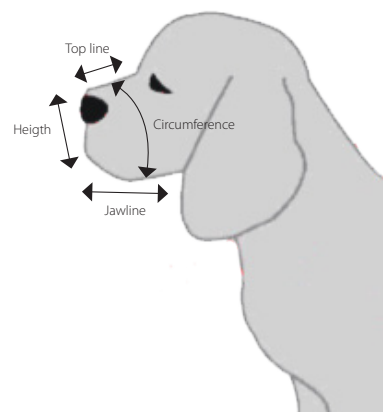

BUSTER Nylon Muzzle for Brachycephalic Breeds

- Covers the eyes
- Restrictive effect
- Allows oral and probe treatments
- Nylon material is X-ray transparent
- Machine washable at 30°C

- | Cat. No | Description |
|---------|---|
| 279460 | BUSTER Vet Nylon muzzle for brachycephale dog, size S |
| 279461 | BUSTER Vet nylon muzzle for brachycephale dog, size L |



BUSTER Nylon Cat Muzzle

- Made of hardwearing but comfortable polypropylene material
- Practical clamp makes it easy to put on the muzzle
- Machine washable at 30°C.

Cat. No	Description
279390	Nylon muzzle for cat, one size



BUSTER Vet Nylon Cat Muzzle

- Covers the eyes
- Restrictive effect
- Allows oral and probe treatments
- Nylon material is X-ray transparent
- Machine washable at 30°C

Cat. No	Description
279462	BUSTER Vet nylon muzzle for cat, size S
279463	BUSTER Vet nylon muzzle for cat, size M
279464	BUSTER Vet nylon muzzle for cat, size L
279459	Set of BUSTER Vet nylon muzzle for cat, size S, M, L
279489	Set of 6 BUSTER Easy-ID Dog Nylon Muzzles XS - 2XL + 1 BUSTER muzzle for cat, size M



BUSTER Plastic Dog Muzzle

- Made of strong, special plastic
- Nose reinforcement
- Strong leather strap

Suitable for dog breeds such as

Size 1	Chihuahua, Japanese Chin, Maltese
Size 2	Papillion, Yorkshire Terrier, Bichon Frisé
Size 3	Standard Dachshund
Size 4	Border Terrier, Cairn Terrier, Cavalier Spaniel, Jack Russell Terrier
Size 5	Bedlington Terrier, Basenji, Fox Terrier
Size 6	Beagle, Border Collie, Cocker Spaniel, Poodle, Welsh Terrier
Size 7	Dalmatian, Elkhound, German Shepherd, Pointer, Springer Spaniel
Size 8	English Setter, Flat Coated Retriever, Foxhound, Golden Retriever
Size 9	Gordon Setter, German Shepherd, Old English Sheepdog, Weimaraner
Size 10	Irish Wolfhound, Afghan Hound, Bouvier



Cat. No	Description	Cat. No	Description
272300	BUSTER Plastic dog muzzle, size 1	272305	BUSTER Plastic dog muzzle, size 6
272301	BUSTER Plastic dog muzzle, size 2	272306	BUSTER Plastic dog muzzle, size 7
272302	BUSTER Plastic dog muzzle, size 3	272307	BUSTER Plastic dog muzzle, size 8
272303	BUSTER Plastic dog muzzle, size 4	272308	BUSTER Plastic dog muzzle, size 9
272304	BUSTER Plastic dog muzzle, size 5	272309	BUSTER Plastic dog muzzle, size 10

BUSTER Plastic Muzzle for Brachycephalic Breeds

- Made of strong, special plastic
- Nose reinforcement
- Strong leather strap

Suitable for dog breeds such as

Size 4	Boxer, Pit Bull Terrier	Cat. No	Description
Size 5	Rottweiler, Bulldog, Mastiff	272311	BUSTER Plastic muzzle for brachycephalic breeds, size 4
		272312	BUSTER Plastic muzzle for brachycephalic breeds, size 5



BUSTER Metal Dog Muzzle

- Robust metal dog muzzle
- Nose reinforcement
- Strong leather strap

Suitable for dog breeds such as

Size 5	Beagle, Border Collie, Cocker Spaniel, Poodle, Welsh Terrier
Size 6	Dalmatian, Elkhound, German Shepherd, Pointer, Springer Spaniel
Size 7	English Setter, Flat Coated Retriever, Foxhound, Golden Retriever
Size 8	Gordon Setter, German Shepherd, Old English Sheepdog, Weimaraner
Size 9	Irish Wolfhound, Afghan Hound, Bouvier, Grand Dane
Size 10	Large Irish Wolfhound, large Afghan Hound, large Bouvier, large Grand Dane

Cat. No	Description
272325	BUSTER Metal dog muzzle, size 5
272326	BUSTER Metal dog muzzle, size 6
272327	BUSTER Metal dog muzzle, size 7
272328	BUSTER Metal dog muzzle, size 8
272329	BUSTER Metal dog muzzle, size 9
272330	BUSTER Metal dog muzzle, size 10

