

# BUSTER Body Sleeves Front Legs

The unique Body Sleeves are available in a version for front legs and a version for hind legs depending on the area affected on the dog.

The Body Sleeves can be used for both cats and dogs. Please see the guidelines for sizes.

## Features and Benefits of the BUSTER Body Sleeves

- Comfortable, soft and flexible
- Freedom of movement
- Easy fitting and removal
- Made from polyester/viscose/elastane

## BUSTER Body Sleeves are perfect for

- Postoperative care
- Wound care
- Spay/neuter recovery
- Hot spots and other skin ailments
- In cases of skin diseases
- Incontinence, urogenital or perianal issues




## BUSTER Body Sleeves Front Legs

- The extra sleeve fabric can be folded around the paw of the dog in case of a paw injury. Please use an extra anti-slid paw protection for the safety of the dog.
- The remaining sleeve fabric can be cut off in a suitable length for the individual dog



# BUSTER Body Sleeves Front Legs

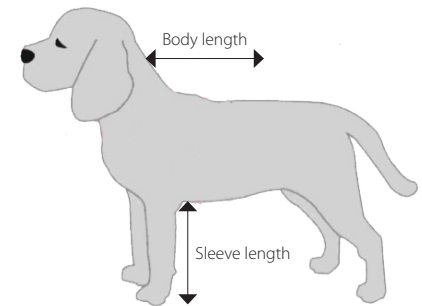

## Guideline for Sizes

The BUSTER Body Sleeves for front legs is available in 8 sizes.

Cat. No	Description	Sleeve Length	Body Length	Suitable for
274046	BUSTER Body Sleeves, front legs XXXS	21 cm	24 cm	Small Chihuahua, kitten
274032	BUSTER Body Sleeves, front legs, XXS	25 cm	30 cm	Chihuahua, Toy poodle, Shih Tzu, Cat
274033	BUSTER Body Sleeves, front legs, XS	29 cm	36 cm	Yorkshire Terrier, Lhasa Apso, Toy poodle, Cat
274034	BUSTER Body Sleeves, front legs, S	33 cm	42 cm	Maltese, Pekingese, French Bulldog, Pug, Terrier
274035	BUSTER Body Sleeves, front legs, M	37 cm	46 cm	English Cocker Spaniel, Samoyed, Portuguese Podengo
274036	BUSTER Body Sleeves, front legs, L	41 cm	50 cm	Schnauzer, Poodle, Fox Terrier, Chow-Chow
274037	BUSTER Body Sleeves, front legs, XL	45 cm	57 cm	Collie, Appenzeller, Boxer, Labrador Retriever
274038	BUSTER Body Sleeves, front legs, XXL	49 cm	64 cm	Rottweiler, Golden Retriever, Herdershond

## Instructions for use

1. Roll up the Body Sleeves between the hands having the KRUUSE logo at the top towards your own body
2. Pull the Body Sleeves over the dog's head having the KRUUSE logo on top of the dog's body
3. Pull the Body Sleeves over the dog's body
4. Roll up one of the sleeves with your hand from the bottom of the sleeve upwards
5. The dog's leg is lifted up and inserted into the sleeve. Your hand inside the sleeve holds the dog's paws avoiding the claws etc. to get caught. With your other hand you pull up the sleeve
6. Pull up the sleeve entirely till the elbow joint of the dog
7. Repeat bullet 4-6 on the second leg
8. Pull up the Body Sleeves around the body
9. Put your hand along the dog's back inside the Body Sleeve, from the back up to the neck, lift up enabling the Body Sleeve to fit correctly into the elbow joint and under the body of the dog

It is important that the sleeves are pulled up well so that the sleeve seam fully stretches the dog's body and snuggles the dog. Adjust the length of the sleeves, the sleeves can be folded to suit the dog or the remaining fabric can be cut off.

The BUSTER Body Sleeves are washable at 40°C



Instructions for use  
Scan QR code to see the video

# BUSTER Body Sleeves Hind Legs

The unique Body Sleeves are available in a version for front legs and a version for hind legs depending on the area affected on the dog.

The Body Sleeves can be used for both cats and dogs. Please see the guidelines for sizes.

## Features and Benefits of the BUSTER Body Sleeves

- Comfortable, soft and flexible
- Freedom of movement
- Easy fitting and removal
- Made from polyester/viscose/elastane

## BUSTER Body Sleeves are perfect for

- Postoperative care
- Wound care
- Spay/neuter recovery
- Hot spots and other skin ailments
- In cases of skin diseases
- Incontinence, urogenital or perianal issues




## BUSTER Body Sleeves Hind Legs

- The extra sleeve fabric can be folded around the paw of the dog in case of a paw injury. Please use an extra anti-slid paw protection for the safety of the dog.
- The remaining sleeve fabric can be cut off in a suitable length for the individual dog



# BUSTER Body Sleeves Hind Legs

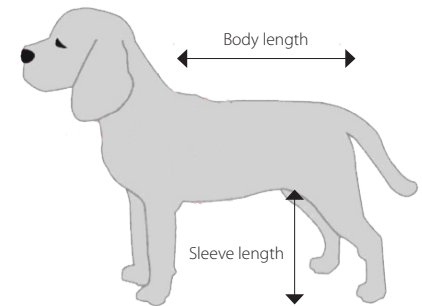

## Guideline for Sizes

The BUSTER Body Sleeves for hind legs is available in 8 sizes.

Cat. No	Description	Sleeve Length	Body Length	Suitable for
274047	BUSTER Body Sleeves, hind legs, XXXS	27 cm	28 cm	Small Chihuahua, kitten
274039	BUSTER Body Sleeves, hind legs, XXS	31 cm	36 cm	Chihuahua, Toy poodle, Shih Tzu, Cat
274040	BUSTER Body Sleeves, hind legs, XS	35 cm	45 cm	Yorkshire Terrier, Lhasa Apso, Toy poodle, Cat
274041	BUSTER Body Sleeves, hind legs, S	39 cm	50 cm	Maltese, Pekingese, French Bulldog, Pug, Terrier
274042	BUSTER Body Sleeves, hind legs, M	43 cm	55 cm	English Cocker Spaniel, Samoyed, Portuguese Podengo
274043	BUSTER Body Sleeves, hind legs, L	47 cm	61 cm	Schnauzer, Poodle, Fox Terrier, Chow-Chow
274044	BUSTER Body Sleeves, hind legs, XL	51 cm	71 cm	Collie, Appenzeller, Boxer, Labrador Retriever
274045	BUSTER Body Sleeves, hind legs, XXL	55 cm	81 cm	Rottweiler, Golden Retriever, Herdershond

## Instructions for use

1. Roll up the Body Sleeve and one sleeve between the hands, the KRUUSE logo must be on top of the dog's back
2. Put one of the dog's leg legs into the sleeve and pull the sleeve entirely till the dog's groin  
Pull the tail through the tail hole at the end of the Body Sleeve
3. Roll up the other sleeve and put the dog's second leg into the sleeve, pull up the sleeve entirely till the groin
4. Drag the Body Sleeve over the body of the dog
5. Lift up one front leg and pull through the leg hole in the Body Sleeve
6. Lift up the second front leg and pull through the leg hole in the Body Sleeve
7. Drag the Body Sleeve all the way up to the neck of the dog. Put your hand inside the Body Sleeve along the dog's back from the dog's ears to the tail - and pull up the Body Sleeve in place

It is important that the sleeves are pulled up all the way so that the seam follows the body of the dog (in the groin).

Adjust the length of the sleeves of the hind legs, the sleeves can be folded to suit the dog or the remaining fabric can be cut off.



Instructions for use  
Scan QR code to see the video

**JØRGEN KRUUSE A/S**  
Havretoften 4  
DK-5550 Langeskov  
Denmark  
Tlf: +45 72 14 14 14  
Fax: +45 72 14 14 00  
info@kruise.com

**KRUUSE SVENSKA AB**  
Ullsväg 30  
756 51 Uppsala  
Sweden  
Tel: +46 (0)18 489 97 00  
Fax: +46 (0)18 489 97 98  
kruise.svenska@kruise.com

**KRUUSE NORGE AS**  
Dyrløkkeveien 19  
N-1440 Drøbak  
Norway  
Tlf: +47 64 90 75 00  
Fax: +47 64 90 75 01  
kruise.norge@kruise.com

**KRUUSE UK LTD.**  
12 Sherburn Network Centre,  
Lancaster Close, Sherburn in Elmet,  
North Yorkshire, LS25 6NS  
United Kingdom  
Phone: +44 01977 681523  
Fax: +44 01977 683537  
kruise.uk@kruise.com

**JØRGEN KRUUSE A/S INTERNATIONAL**  
Havretoften 4  
DK-5550 Langeskov  
Denmark  
Tel: +45 72 14 15 16  
Fax: +45 72 14 15 00  
export@kruise.com

**KRUUSE POLSKA SP. Z O.O**  
Ul. Knapowskiego 12/1  
60-126 Poznan  
Poland  
Tel. +48 61 22 22 160  
Fax. +48 61 22 22 161  
kruise.polska@kruise.com