

KRUUSE Vet-Flex

Superior quality of flexible cohesive bandage for fixation of dressings and controlled support of sprains and distortions.

Strong, comfortable and user-friendly. Easy to tear.

- High quality flexible bandage
- Easy to tear
- Skin friendly
- Minimal shrinkage



Features and benefits

Reliable and stable fixation

The high weight of the material, 35 g/m², makes the bandage strong and firm, yet comfortable. Gives good compression.

Easy to tear

No usage of scissor, as it is possible to tear in a straight line.

Skin friendly

KRUUSE Vet-Flex is coated with latex, which makes the cohesiveness stick to it-self, but not to other surfaces like skin or hair. The bandage is also porous for transition of air.

Very flexible

The KRUUSE Vet-Flex can be stretched to almost double the unrolled length; therefore it is possible to bandage all kind of injuries of the animal body and control the compression.

Very good elasticity

The bandage has spandex threads, which gives a suitable shrinkage, that is necessary to give a firm compression and fixation. Will not constrict unless wrongly applied.

Easy to remove

The features of the KRUUSE Vet-Flex cohesive bandage make the bandage very easy to remove even without a scissor.

Hygienic

Water resistant, excellent to keep bandage dry from the outside. Each roll is wrapped individually in an easy to tear pouch.

Good compliance from the patient

The bandage is very light, does not constrict and is not sticky to touch.



Easy to tear



Latex threads



Spandex threads

Recommended use for KRUUSE Vet-Flex

- Controlled support and compression of joints, sprains, distortions
- Fixating dressings

Instructions for use

Application

Fixate the end of the bandage and apply evenly over a larger area of the injured body part stretching the bandage according to the level of compression needed.

When application is done, do a straight tear of the bandage. . . .

Do not twist; it will make the tearing more difficult.

When applied, squeeze gently the layers together to enhance the cohesiveness.

Use KRUUSE Vet-Flex as secondary/tertiary bandage on top of primary/secondary dressing.

Primary dressing suggestion

- KRUUSE Hydro Gel
- KRUUSE Manuka Honey

Secondary dressing suggestion

- KRUUSE Foam Dressing, which can also be used with
- KRUUSE Vet-Plast
- KRUUSE Soft-Flex
- BUSTER Tubular Bandage

Removal

Unroll bandage or cut gently with a bandage scissor, be careful not to cut into the skin.

Do not use

If patient or user is allergic to latex.

Be aware of compression during the application, if the bandage is applied with too much tension or with too many layers over a small part of the body. Apply evenly over a larger area.



Straight tear



Do not twist



Squeeze gently

Cat. No	Description
160728	KRUUSE Vet-Flex 2.5 cm x 4.5 m, red, 10/pk
160729	KRUUSE Vet-Flex 5 cm x 4.5 m, white, 10/pk
160730	KRUUSE Vet-Flex 5 cm x 4.5 m, red, 10/pk
160731	KRUUSE Vet-Flex 7.5 cm x 4.5 m, green, 10/pk
160732	KRUUSE Vet-Flex 7.5 cm x 4.5 m, white, 10/pk
160733	KRUUSE Vet-Flex 10 cm x 4.5 m, white, 10/pk
160734	KRUUSE Vet-Flex 10 cm x 4.5 m, red, 10/pk
160735	KRUUSE Vet-Flex 10 cm x 4.5 m, blue, 10/pk
160736	KRUUSE Vet-Flex 10 cm x 4.5 m, green, 10/pk
160737	KRUUSE Vet-Flex 10 cm x 4.5 m, yellow, 10/pk
160738	KRUUSE Vet-Flex 10 cm x 4.5 m, black, 10/pk
160739	KRUUSE Vet-Flex 10 cm x 4.5 m, ass. of neon colors, 10/pk
160742	KRUUSE Vet-Flex 15 cm x 15 m, red, 4/pk
160743	KRUUSE Vet-Flex 15 cm x 15 m, blue, 4/pk

For veterinary use only