

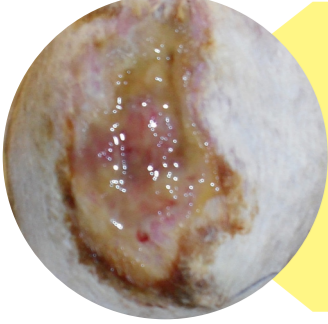

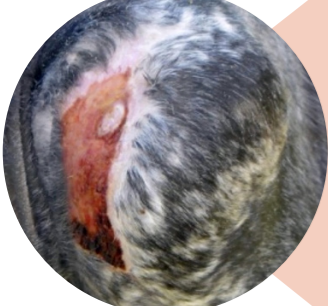


KRUUSE Equine Wound Chart



	Dry			Moderate exudating			Heavily exudating		
	Primary	Secondary	Tertiary	Primary	Secondary	Tertiary	Primary	Secondary	Tertiary
 <p>► Acute/traumatic wound - Evaluate the wound - Clean the wound using KRUUSE wound lavage system - Dress and bandage according to evaluation</p>									
 <p>► Necrotic - Debride the necrotic tissue and lavage - Protect wound margins and surrounding skin from exudate</p>	KRUUSE Manuka G KRUUSE Manuka ND KRUUSE HydroGel	KRUUSE Foam KRUUSE Vet-Line	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage	KRUUSE Manuka G KRUUSE Manuka AD KRUUSE HydroGel	KRUUSE Foam Steri-Protect KRUUSE Vet-Line	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage	KRUUSE Manuka G KRUUSE Manuka AD	KRUUSE Foam Steri-Protect	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage
 <p>► Sloughy - Debride devitalised tissue and lavage - Protect wound margins and surrounding skin from exudate</p>	KRUUSE Manuka G KRUUSE Manuka ND KRUUSE HydroGel	KRUUSE Foam KRUUSE Vet-Line	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage	KRUUSE Manuka G KRUUSE Manuka AD KRUUSE HydroGel	KRUUSE Foam Steri-Protect KRUUSE Vet-Line	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage	KRUUSE Manuka G KRUUSE Manuka AD	KRUUSE Foam Steri-Protect	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage
 <p>► Granulating - Maintain a moist environment and protect new healthy granulation from damage between dressing changes</p>	KRUUSE HydroGel	KRUUSE Foam KRUUSE Vet-Line	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage	KRUUSE Manuka G KRUUSE Manuka ND KRUUSE HydroGel	KRUUSE Foam KRUUSE Vet-Line	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage			
 <p>► Epithelialising - Maintain a clean environment and protect from unnecessary trauma</p>	KRUUSE HydroGel	KRUUSE Foam KRUUSE Vet-Line	KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage						