

BUSTER Body Suit for Dogs & Cats

The soft and comfortable BUSTER Body Suit is used as protection and comfort after surgery or in case of skin disease. It is an ideal solution for patients recovering from surgery or minor injuries or have tender areas caused by excessive licking, biting or chewing.

BUSTER Body Suit is Perfect for

- Post operative care
- Wound care
- Spay/neuter recovery
- Hot spots and other skin ailments
- In cases of skin diseases
- Incontinence, urogenital or perianal issues
- In season

BUSTER Body Suit Advantages

- Comfortable, soft and flexible
- Freedom of movement
- Easy fitting and removal
- Press studs for more security
- Made from cotton/elastane

**DOUBLE FABRIC
UNDER THE ABDOMEN**



The BUSTER Body Suit is available in two versions for dogs and one version for cats.


Body Suit EasyGo for Cats

Double fabric under the abdomen enabling pads to be inserted in case of fluid loss.

Body Suit EasyGo for cats with extended tail opening.

The EasyGo version helps the cat when nature calls.

It is suitable for female cats and tomcats.

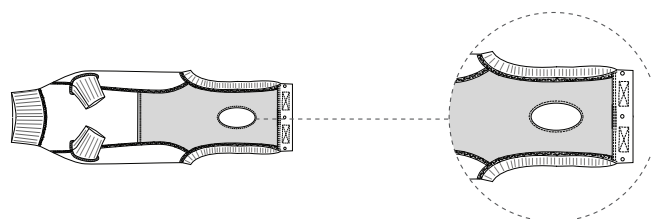


Body Suit EasyGo for Dogs

Body Suit EasyGo for dogs with extended tail opening and extra slit.

The EasyGo version helps the dog when nature calls.

It is suitable for bitches and male dogs.

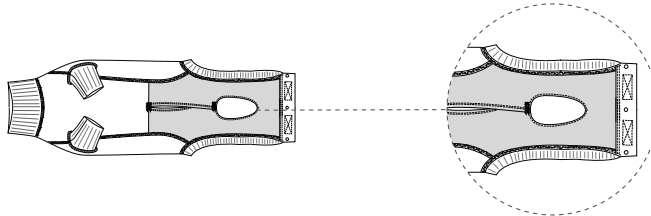


Body Suit Classic for Dogs

Double fabric under the abdomen enabling pads to be inserted in case of fluid loss, incontinence or when the animal is in season.

Undo press studs and touch fasteners and pull back the fabric

and roll up the fabric under the abdomen and above the back when nature calls.

It is suitable for bitches and male dogs

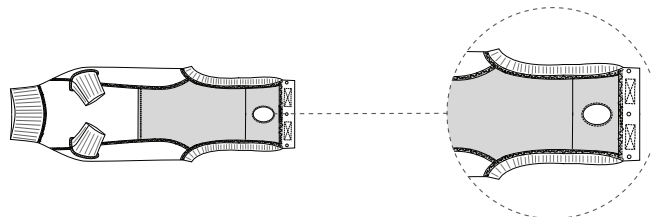

Instructions for Use

Step 1


- Insert the head into the Body Suit
- Insert each front leg separately into the openings provided

Step 2

- Pull the Body Suit as far as possible over the body

Step 3

- Put the closing flap between the rear legs and insert the tail through the tail opening
- Attach the closing flap on the velcro tabs on the loin and secure the press studs

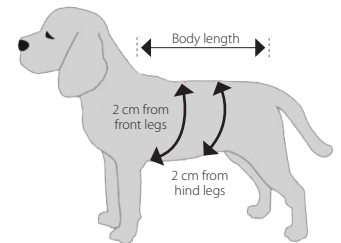


Guideline for Sizes

The BUSTER Body Suit EasyGo for dogs is available in 9 sizes. Size measured from collar to base of the tail.

| Cat. No | Size | Body length | Chest circumference (2 cm from front legs) | Abdomen circumference (2 cm from hind legs) | Suitable for |
|---------|------|-------------|--|---|--|
| 273951 | XXXS | 25 cm | 15 cm | 18 cm | Small Chihuahua |
| 273952 | XXS | 32 cm | 19 cm | 17 cm | Chihuahua, Toy poodle |
| 273953 | XS | 38 cm | 23 cm | 21 cm | Yorkshire, Terrier, Lhasa, Toy poodle |
| 273954 | S | 41 cm | 27 cm | 25 cm | Malteser, Pekingeser, French Bulldog, Mops, Terrier |
| 273955 | M | 45 cm | 29 cm | 26 cm | English Cocker Spaniel, Samojeed, Portuguese Podengo |
| 273956 | L | 52 cm | 31 cm | 28 cm | Schnauzer, Beagle, Poodle, Fox Terrier, Chow-Chow |
| 273957 | XL | 62 cm | 35 cm | 32 cm | Collie, Appenzeller, Boxer, Schnauzer, Labrador |
| 273958 | XXL | 73 cm | 39 cm | 35 cm | Rottweiler, Golden Retriever, Herdershund |
| 273959 | XXXL | 75 cm | 43 cm | 38 cm | Saint Bernhard, Newfoundlander |

The BUSTER Body Suit Classic for dogs is available in 9 sizes. Size measured from collar to base of the tail.

| Cat. No | Size | Body length | Chest circumference (2 cm from front legs) | Abdomen circumference (2 cm from hind legs) | Suitable for |
|---------|------|-------------|--|---|--|
| 273968 | XXXS | 26 cm | 18 cm | 15 cm | Small Chihuahua |
| 273969 | XXS | 33 cm | 21 cm | 18 cm | Chihuahua, Toy poodle |
| 273970 | XS | 39 cm | 23 cm | 20 cm | Yorkshire, Terrier, Lhasa, Toy poodle |
| 273971 | S | 42 cm | 26 cm | 23 cm | Malteser, Pekingeser, French Bulldog, Mops, Terrier |
| 273972 | M | 46 cm | | | English Cocker Spaniel, Samojeed, Portuguese Podengo |
| 273973 | L | 53 cm | | | Schnauzer, Beagle, Poodle, Fox Terrier, Chow-Chow |
| 273974 | XL | 63 cm | | | Collie, Appenzeller, Boxer, Schnauzer, Labrador |
| 273975 | XXL | 74 cm | | | Rottweiler, Golden Retriever, Herdershund |
| 273976 | XXXL | 76 cm | | | Saint Bernhard, Newfoundlander |

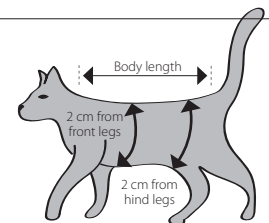

| Cat. No | Size | Body length | Chest circumference (2 cm from front legs) | Abdomen circumference (2 cm from hind legs) | Suitable for |
|---------|------|-------------|--|---|--------------------------|
| 273960 | XXXS | 27.5 cm | 18 cm | 15 cm | Young cat (small kitten) |
| 273961 | XXS | 33 cm | 21 cm | 18 cm | Young cat (kitten) |
| 273962 | XS | 38.5 cm | 23 cm | 20 cm | Adult cat |
| 273963 | S | 43.5 cm | 26 cm | 23 cm | Adult cat (large breed) |

The BUSTER Body Suit is washable at 40°C

Developed for veterinary use by KRUUSE in Denmark
Made in China