

# BUSTER Body Suit for Dogs & Cats

The soft and comfortable BUSTER Body Suit is used as protection and comfort after surgery or in case of skin disease. It is an ideal solution for patients recovering from surgery or minor injuries or have tender areas caused by excessive licking, biting or chewing.

## BUSTER Body Suit is Perfect for

- Post operative care
- Wound care
- Spay/neuter recovery
- Hot spots and other skin ailments
- In cases of skin diseases
- Incontinence, urogenital or perianal issues
- In season

## BUSTER Body Suit Advantages

- Comfortable, soft and flexible
- Freedom of movement
- Easy fitting and removal
- Press studs for more security
- Made from cotton/elastane

**DOUBLE FABRIC  
UNDER THE ABDOMEN**



The BUSTER Body Suit is available in two versions for dogs and one version for cats.


## Body Suit EasyGo for Cats

Double fabric under the abdomen enabling pads to be inserted in case of fluid loss.

Body Suit EasyGo for cats with extended tail opening.

The EasyGo version helps the cat when nature calls.

It is suitable for female cats and tomcats.

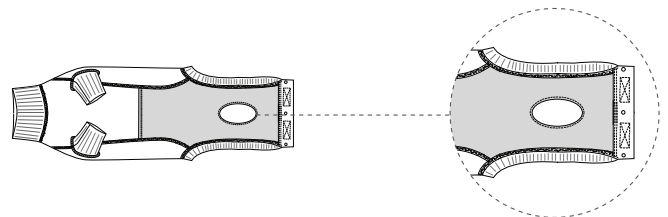


## Body Suit EasyGo for Dogs

Body Suit EasyGo for dogs with extended tail opening and extra slit.

The EasyGo version helps the dog when nature calls.

It is suitable for bitches and male dogs.

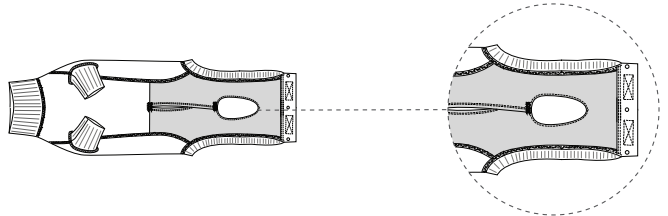


## Body Suit Classic for Dogs

Double fabric under the abdomen enabling pads to be inserted in case of fluid loss, incontinence or when the animal is in season.

Undo press studs and touch fasteners and pull back the fabric and roll up the fabric under the abdomen and above the back when nature calls.

It is suitable for bitches and male dogs

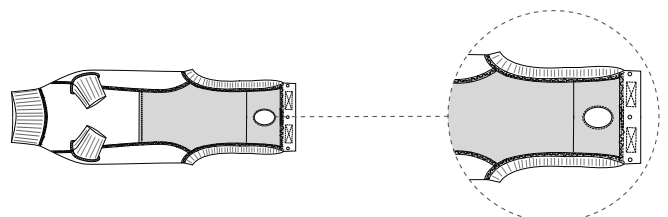

**Instructions for Use**

**Step 1**


- Insert the head into the Body Suit
- Insert each front leg separately into the openings provided

**Step 2**

- Pull the Body Suit as far as possible over the body

**Step 3**

- Put the closing flap between the rear legs and insert the tail through the tail opening
- Attach the closing flap on the velcro tabs on the loin and secure the press studs

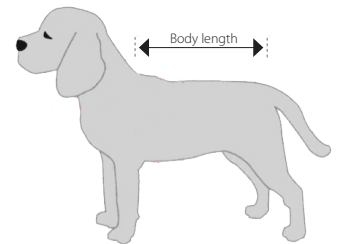


**Guideline for Sizes**

The BUSTER Body Suit EasyGo for dogs is available in 9 sizes.  
Size measured from collar to base of the tail.

Cat. No	Size	Body length	Suitable for
273951	XXXS	25 cm	Small Chihuahua
273952	XXS	32 cm	Chihuahua, Toy poodle
273953	XS	38 cm	Yorkshire, Terrier, Lhasa, Toy poodle
273954	S	41 cm	Malteser, Pekingese, French Bulldog, Mops, Terrier
273955	M	45 cm	English Cocker Spaniel, Samojeed, Portuguese Podengo
273956	L	52 cm	Schnauzer, Beagle, Poodle, Fox Terrier, Chow-Chow
273957	XL	62 cm	Collie, Appenzeller, Boxer, Schnauzer, Labrador
273958	XXL	73 cm	Rottweiler, Golden Retriever, Herdershond
273959	XXXL	75 cm	Saint Bernhard, Newfoundland

The BUSTER Body Suit Classic for dogs is available in 9 sizes.  
Size measured from collar to base of the tail.

Cat. No	Size	Body length	Suitable for
273968	XXXS	26 cm	Small Chihuahua
273969	XXS	33 cm	Chihuahua, Toy poodle
273970	XS	39 cm	Yorkshire, Terrier, Lhasa, Toy poodle
273971	S	42 cm	Malteser, Pekingese, French Bulldog, Mops, Terrier
273972	M	46 cm	English Cocker Spaniel, Samojeed, Portuguese Podengo
273973	L	53 cm	Schnauzer, Beagle, Poodle, Fox Terrier, Chow-Chow
273974	XL	63 cm	Collie, Appenzeller, Boxer, Schnauzer, Labrador
273975	XXL	74 cm	Rottweiler, Golden Retriever, Herdershond
273976	XXXL	76 cm	Saint Bernhard, Newfoundland

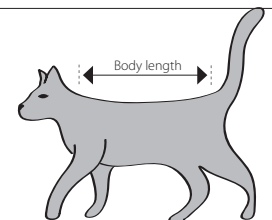

The BUSTER Body Suit EasyGo for cats is available in 4 sizes.  
Size measured from collar to base of the tail.

Cat. No	Size	Body length	Suitable for
273960	XXXS	27.5 cm	Young cat (small kitten)
273961	XXS	33 cm	Young cat (kitten)
273962	XS	38.5 cm	Adult cat
273963	S	43.5 cm	Adult cat (large breed)

The BUSTER Body Suit is washable at 40°C

Developed for veterinary use by KRUUSE in Denmark  
Made in China